


Information sheet MPS-Head Cover®

www.medicalpetshirts.com



NOW ALSO AVAILABLE WITH IMPROVED AIR CIRCULATION:

Due to high demand, we have fast-tracked the production of our new and improved design. So the improved MPS-Headcover® is now available in our warehouse! By using breathable Kool-tex fabric on the light blue inner layer and Head Cover pads, the MPS-Headcovers® are now much more breathable. The result? Even better air circulation! Which is very important for ear infections and healing wounds.

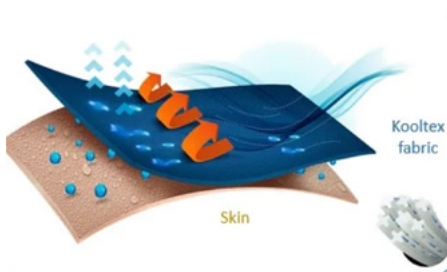


Why use the MPS-Head Cover®?

The MPS-Head Cover® is an alternative and comfortable solution for dogs that need head and/or ear protection for wounds, skin issues, or bandages. This eliminates the need to replace worn out bandages daily, reducing additional stress for the dog. Ideal for an aural hematoma. *Discover how it's now used for MRI safety. Learn more on our blog: <https://medicalpetshirts.com/blogs/vet-mri-safety-hearing-protection/>*

MPS-Head Cover® Benefits

- ✓ Protection for head and ear problems
- ✓ Prevents ear flapping
- ✓ Easily adjustable for a perfect fit
- ✓ Patented NEW breathable design
- ✓ Available in 5 different sizes



MPS-Head Cover®						
Size	Head	Neck	Length	Pads		
MPS Head Cover-XS	30 - 36 cm 11.8- 14.1"	32 - 46 cm 12.6 - 18.1"	17 - 24 cm 6.7 - 9.4"	Small		
MPS Head Cover-S	32 - 40 cm 12.6 - 15.7"	38 - 50 cm 15.0 - 19.7"	19 - 28 cm 7.5 - 11.0"	Small		
MPS Head Cover-M	38 - 46 cm 14.9 - 18.1"	38 - 54 cm 15.0 - 21.1"	21 - 32 cm 8.3 - 12.5"	Small		
MPS Head Cover-L	38 - 50 cm 14.9 - 19.7"	42 - 62 cm 16.5 - 24.4"	21 - 33 cm 8.3 - 12.9"	Medium		
MPS Head Cover-XL	42 - 54 cm 16.5 - 21.3"	46 - 66 cm 18.1 - 25.9"	21 - 34 cm 8.3 - 13.3"	Medium		
MPS HC Pad -S Set	Suitable for sizes XS - M					
MPS HC Pad -M Set	Suitable for sizes L - XL					

How to put the MPS-Head Cover® on the dog?			
Step one	Step two	Step three	Step four
Optional pads: for extra soft protection, you can first insert the MPS-Head Cover Pads® in the MPS-Head Cover®. Open the pocket, take a pad and place the pad's Velcro® on the Velcro® in the pocket of the Head Cover®.	Head closing strap: place the Head Cover® on the dog's head and close it by fastening the Velcro® straps and popper (snap) button. Use at least two rows of snaps for a good fixation.	Neck closing strap: close it by fastening the Velcro® straps and popper (snap) buttons. Use at least two rows of popper buttons.	Optional neck length: if needed, you can adjust the neck length for a better fit by adjusting the Velcro® strap on the back of the neck.

Information sheet MPS-Head Cover®

www.medicalpetshirts.com



FEATURES OF THE MPS-HEAD COVER®

		1. Special pockets for pads The Head Cover® comes with two specially designed pockets on the inside. These pockets hold the MPS-Head Cover Pads® (available separately) for extra soft protection. The updated pads are breathable and feature Kooltex fabric.
		2. Flexible supporting ribs These internal flexible plastic ribs prevent sagging as much as possible and support the Head Cover® in protecting wounds and/or bandages.
		3. Elastic cord The Head Cover® comes with an elastic cord to ensure that the front of the Head Cover® closes around the dog's head as much as possible.
		4. Adjustable fastening straps The Head Cover® is fully closed by way of adjustable fastening straps with Velcro® and snaps. The snaps ensure that the Head Cover® fits as snugly as possible.
		5. Velcro® strip A Velcro® strip can be found at the back for shortening the skull length for a tighter fit, if necessary.

*Note: Make sure not to fasten the Head Cover® too tightly, as this may otherwise lead to breathing and (blood) circulation problems.

PS-Head Cover® Washing instructions			MPS-HC - pads	
Remove the ribs from the Head Cover before washing..	Washing, maintaining and handling of delivered items always remains the responsibility of the customer.			

Do you have any questions about the MPS®-Head Cover?
Contact us via info@medicalpetshirts.com - We are happy to help you!

